




## Post-op wound care (ProntoVet® / Askina®)

It's in your hands – Infection prevention in  
veterinary medicine



# Wound

## Post-op wound care

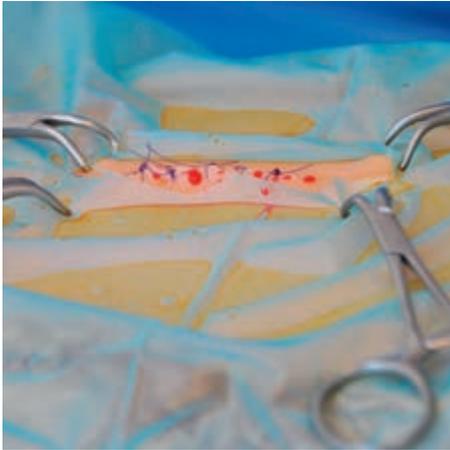


After wound closure, remove the dressing.



Clean the wound with ProntoVet® wound irrigation solution.




After one minute, dry the periphery with a sterile swab so that the wound dressing (Askina® Soft) can adhere properly.

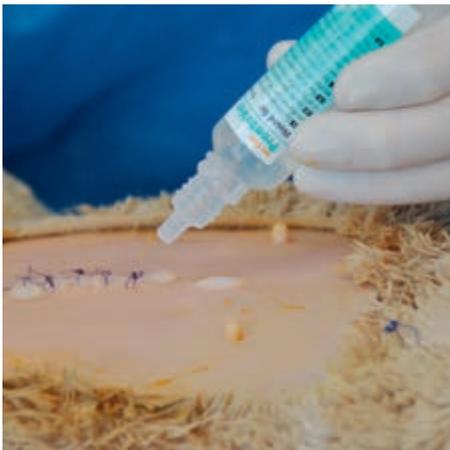

The wound will remain permeable to bacteria for 24 hours. As such, it is advisable to apply ProntoVet® wound gel to the wound.



After that, apply a sterile Askina® Soft wound dressing.

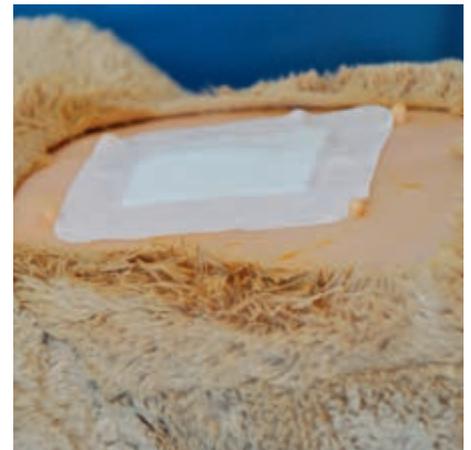

For optimum protection against contamination, keep the wound bandaged for at least 24-48 hours.



Additional information, this poster, and much more can be found at:  
[www.bbraun-vetcare.com/inyourhands](http://www.bbraun-vetcare.com/inyourhands)

B. Braun Vet Care GmbH | Am Aesculap-Platz | 78532 Tuttlingen | [info@bbraun-vetcare.de](mailto:info@bbraun-vetcare.de) | [www.bbraun-vetcare.com](http://www.bbraun-vetcare.com)

The product trademarks "Askina" and "ProntoVet" are registered trademarks of B. Braun Melsungen AG. Subject to technical alterations. This brochure may only be used as information about our products. Reproduction in whole or in part is not permitted.

D-VE23017 1123/PDF/1