


AESCULAP® AdTec® BIPOLAR


DISASSEMBLY | ASSEMBLY INSTRUCTIONS



DISASSEMBLY

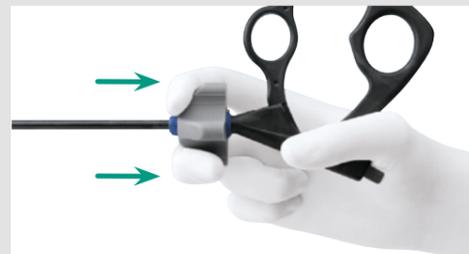


Disconnect shaft and handle
Pull the rotation knob back and hold it in this position.

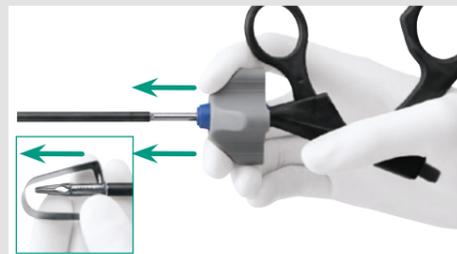


Remove shaft from handle
Hold the jaw insert firmly by its protecting cap and pull it out of the handle.
Important: Do not hold on to the movable handle component.



Disassemble handle
Pull the grey rotation knob forwards off the handle.

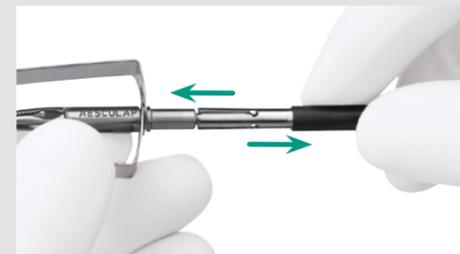


Disassemble shaft I
Hold the jaw insert at the protecting cap and remove the outer insulation tube.

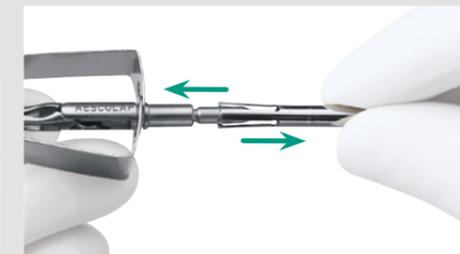

Disassemble shaft II
Hold the jaw insert at the protecting cap and remove the inner metal tube.


ASSEMBLY

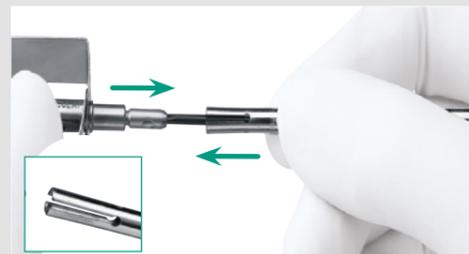


Assemble shaft I
Slide the metal inner tube onto the jaw insert with its four spring tongues towards the working end until the tongues snap in place.



Assemble shaft II
Slide the outer insulation tube as far as it will go onto the inner metal tube.

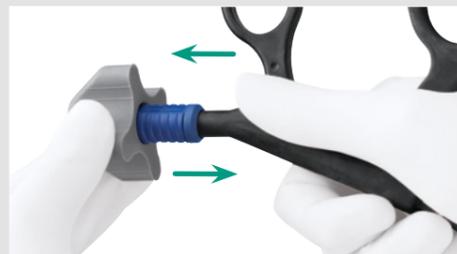

Prepare handle
Push the grey rotation knob onto the handle from the front. Bring the movable handle component into its forward position.

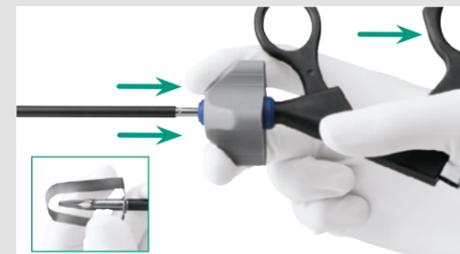

Connect shaft and handle
Pull the rotation knob back and hold it. Hold the shaft firmly by its protecting cap and push it into the handle. Release the rotation knob, which moves forward.
Important: Do not hold on to the movable handle component.



Function test
Test the assembly of the instrument by opening and closing the handle to ensure the jaw is opening and closing.

CARE




Lubricate the defined places on the instruments with AESCULAP® STERILIT® Oil after each cleaning cycle.



PROCESSING & ACCESSORIES

- Reprocess the product immediately after use
- The product can be sterilized either in disassembled or in assembled condition
- For advice on cleaning, disinfection and sterilization, please follow the instructions for use. For effective cleaning, particularly of the jaws, we recommend B. Braun Stabimed®.
- For the latest information on reprocessing, please also refer to the Aesculap Extranet via www.aesculap-extra.net
- For accessories and spare parts, please refer to Brochure C75802 or visit www.endoscopy-catalog.com

REPAIRS

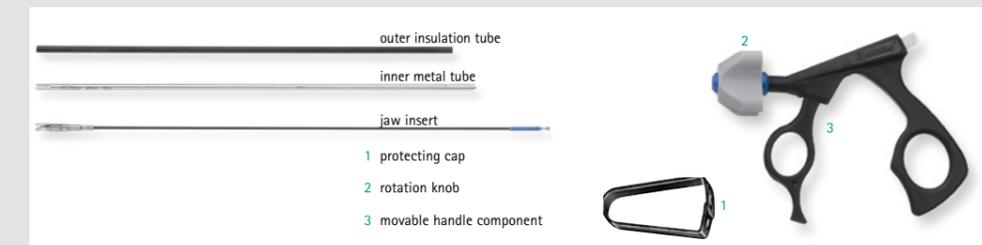

- In order to ensure the preservation of full guarantee and warranty rights, only persons authorized by Aesculap may carry out repairs. Please send articles for repair to: Aesculap Technical Service, Am Aesculap-Platz, D-78532 Tuttlingen, Germany. Additional service addresses may be obtained from the same address or via ats@aesculap.de