Uro-Tainer®

Solutio R Twin / Suby G Twin

PVC-free

BG	Информация за употреба	2
CZ	Informace pro použití	4
DE	Benutzerinformationen	6
DK	Oplysninger om brug	8
EE	Kasutusjuhend	10
EL	Πληροφορίες χρήσης	12
ES	Instrucciones de uso	14
FI	Tietoa käyttäjälle	16
FR	Mode d'emploi	18
GB	Information for Use	20
HR	Informacije za korisnika	22
HU	Használati utasítás	24
IT	Istruzioni per l'uso	26
NL	Informatie voor gebruik	28
NO	Informasjon om bruk	30
PL	Instrukcja obsługi	32
PT	Instruções de utilização	34
RO	Instrucţiuni de utilizare	36
SE	Bruksanvisning	38
SK	Informácie o použití	40

Solutio R Twin / Suby G Twin

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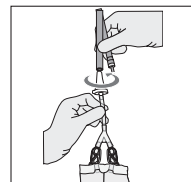
Solutio R Twin / Suby G Twin

Maintenance solution for urethral and suprapubic catheters

1 Introduction

Uro-Tainer® Maintenance solutions are a ready-to-use, disposable systems for the maintenance of indwelling urethral and suprapubic catheters. Each Uro-Tainer® Maintenance solution is overwrapped in a protective bag and is sterilised.

2 Product profile and areas for use:

- a) Uro-Tainer® 0.9% NaCl: For routine rinsing of catheters (removal of small blood clots, tissue, debris).
- b) Uro-Tainer® Solutio R: Dissolution of persistent encrustation, treatment of encrusted catheter to reduce trauma during removal of catheter
- c) Uro-Tainer® Suby G: Dissolution of encrustation

3 Intended users

Medical professional users or patients trained by Medical professional to perform the maintenance themselves. Restricted to adults.

4 Possible side effects

Citric acid may cause some patients to experience slight irritation, discomfort, and even temporary pain, a burning sensation, or spasms of the bladder (desire to urinate). If these effects occur, the less concentrated solution Suby G is recommended, carry out the instillation less frequently, or alternate it with an instillation of NaCl 0.9 %, because this solution is not known to cause such effects.

5 Restrictions of use

Uro-Tainer® Maintenance solutions should not be used in case of hypersensitivity (allergy) to listed ingredients of the solution. Furthermore, the products should not be used in case cystitis or other uro-genital condition that can produce haematuria (blood in the urine). Uro-Tainer® Maintenance solutions should not be used several days after surgery, (depends on the healing process and must be decided by the attending physician) on the bladder or the urinary tract and should not be used for children and newborns

6 Warnings and precautions

- Talk to your doctor, pharmacist or nurse before using Uro-Tainer®.
- Under no circumstances should the solution be injected or infused.
- Instill the solution by gravity feed, avoid force.
- Apply using aseptic technique. A port disinfection before use is recommended.
- Only to be used if the bag and the sealing cap are intact, and the solution is clear.
- This is a single-use system. After opening, it must be used and disposed of promptly. Re-use of such devices creates a potential risk of patient or user. It may lead to contamination and/or limited functionality of the device, which causes to injury, illness or death of the patient.
- Discard properly after use.
- If the solution leaks during the flushing process from the urethra, the catheter is not correctly placed in the bladder. In that case the catheter needs to be replaced by a new one.
- If the solution is not flowing, it may be the catheter is bent or plugged. In that case the catheter needs to be replaced by a new one.
- Precautions should be taken when operating on patients with spinal injuries because of the risk of autonomic dysreflexia.

7 How to store Uro-Tainer®

Do not use the Uro-Tainer® after the expiry date which is stated on the outer plastic bag. Do not refrigerate or freeze. Store in the original package. Only use clear solutions in undamaged bottles. After opening, the Uro-Tainer® must be used and disposed of promptly.

8 Composition

- a) Uro-Tainer® 0.9% NaCl
100 ml solution contains: Sodium chloride 0.9 g in Water for Injections
- b) Uro-Tainer® Solutio R
100 ml solution contains: Citric monohydrate acid 6.0 g, Gluconolactone, Light magnesium carbonate, Disodiumedetate dihydrate in Water for Injections
- c) Uro-Tainer® Suby G
100 ml solution contains: Citric monohydrate acid 3.23 g, Light magnesium oxide, Sodium bicarbonate, Disodiumedetate dihydrate in Water for Injections

9 Application and instruction for use

- Uro-Tainer® are closed and sterile systems to be connected to the urinary catheter and work all according to the operating instructions on the next page.
- The Uro-Tainer® sachet is supplied sterile and should not be removed from its overwrapping until required for use.
- If desired, bring the solution up to body temperature by immersing the wrapped sachet in warm water.
- The bag that contains the rinsing solution collects the liquid after rinsing of the catheter. Catheter rinsing and liquid collecting after rinsing are performed by gravity alone, by maintaining the device above and below the level of the bladder respectively.
- Uro-Tainer® 0.9 % NaCl, acting with a mechanical rinsing effect, can be administered as "in and out" application, while Uro-Tainer® Suby G and Solutio R, based on acidic solutions, need to be administered into the catheter and left there for 5 minutes to fully exploit their chemical action. This process is recommended 1 to 2 times per day.
- In general, the daily use is only indicated for postoperative patients during a short period. In contrast to that, long-time catheterized patients have catheters for very long periods. In this case the catheter is only rinsed with Uro-Tainer® 2-3 times per week.
- If you use Uro-Tainer® more often than you should, no adverse events are expected.
- Pay particular attention to hygienic handling of the urinary catheter.
- Never act independently without consulting your ambulatory care service or doctor

10 Appearance

The solution appears as a clear and colorless liquid. Uro-Tainer® are made of a soft bag free of poly(vinylchloride) (PVC) welded with a flexible PVC-free tube and a urologic closure. The bag material used is a colorless and transparent pharmaceutical film. Uro-Tainer® are available in package sizes of 50 ml, and 100 ml.

11 Shelf life

According to the expiry date; store at room temperature.

12 Disposal of the device

There are no special requirements for disposal of Uro-Tainer® Maintenance solutions other than to observe applicable laws, standards, and guidelines for the disposal of medical waste. Packaging materials must also be disposed of in compliance with applicable national requirements.

13 Serious incident

The user and/or patient should report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

14 Use together with other devices

Catheter A

2-way Foley catheter, latex-free silicone, Balloon 30 cc, 18 Ch, hydrogel coating

Catheter B

Straight whistle tip catheter, silicone 40 cm, 18 Ch

Catheter C

Bladder catheter without balloon, PVC, 37 cm, 18 Ch, latex-free

Catheter D

Transurethral Foley catheter, 2 -way, latex-free silicone, 41 cm, 18 Ch

Catheter E

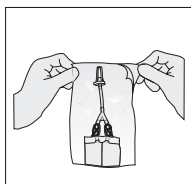
Transurethral Foley Nelaton balloon catheter, latex-free silicone, 41 cm, 18 Ch

This leaflet was last revised in 06/2024.


1.

Peel open the plastic overwrap and remove the Uro-Tainer® Twin.


2.

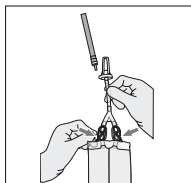Close both tubes with the green and white clamps.


3.

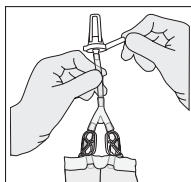Remove the tear away closure ring.


4.

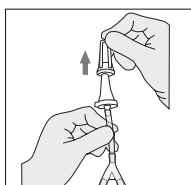Twist and withdraw cap without touching sterile connector. Release the white clamp into the open position and allow a few drops of the solution to flow into the catheter to remove any air locks.


5.

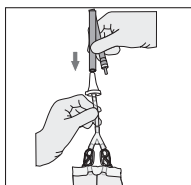Insert connector into the catheter.


6.

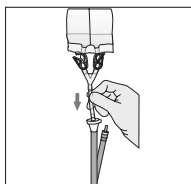Holding Uro-Tainer® Twin above the level of the bladder, allow the solution from the 1st chamber to flow into the bladder. To avoid any potential trauma to the bladder do not use force or agitation.


7.

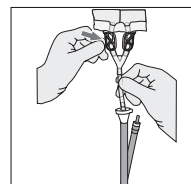Close open tube with white clamp.


8.

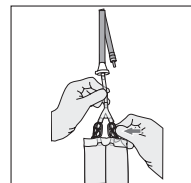After 5 minutes release white clamp and hold bag below the level of the bladder, and drain the solution. Close tube with white clamp.


9.

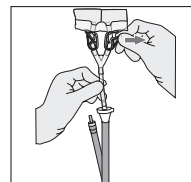Now repeat the process using the 2nd chamber of solution. Hold Uro-Tainer® Twin above the level of the bladder and release the green clamp. Allow the solution to flow into the bladder. Again, do not use force or agitation.


10.

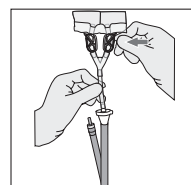Close open tube with green clamp.


11.

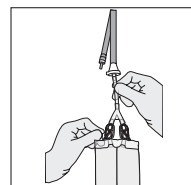After 5 minutes release green clamp and hold bag below the level of the bladder, and drain the solution. Close tube with green clamp. Ensure both clamps are now closed.

12.

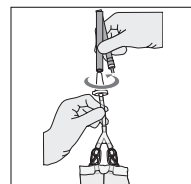Remove the connector from the catheter by turning gently. Now connect the catheter to a sterile drainage bag. Discard after treatment. For single use only.

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B. Braun Australia Pty Ltd
B. Braun Melsungen AG
B. Braun Medical AG